

- 1.** Before using PlastiSept eco for the first time, clean all the surfaces of your equipment with warm water and neutral detergent to remove the residues of previous disinfectants and bring the surfaces to the neutral PH.
- 2.** Empty and clean all the bottles, used for storage and dispensing of the previous disinfectants.
- 3.** Never mix Plastisept eco with another type of disinfectant.
- 4.** Never apply PlastiSept eco directly on the surfaces but use it in conjunction with Maxi wipes or spray on the cloth of your choice prior to the application on the surfaces.
- 5.** Use very little of the product! If the product does not evaporate within max 1 min, you have used too much of it.
- 6.** Use pre-soaked PlastiSept eco wipes if possible, to avoid the confusion of how much is enough or too much.
- 7.** Use one wipe per area, example one wipe per chair etc. not more than 3-4 per surgery room after each patient.
- 8.** Periodically, we recommend min once every 2 weeks, perform the full wash of all the equipment, using neutral soap and warm water to strip the surfaces from accumulation of active ingredients on the surfaces.