

Decontamination of procedural water lines in medical and dental units

as at: 15 Nov. 2018 - AU

ARTG No: 226413

Liquid concentrate for the continuous maintenance of the water quality in procedural water lines (including instruments and tumbler filler) of medical and dental units.



- Application of concentrate at 1 %
- Reduces germ load and prevents the regrowth of germs
- Removes and prevents the formation of algae (biofilm)
- Prevents the sedimentation of lime and deposits
- Non-corrosive

Composition

Phenoxyethanol, PHMB (biguanide), EDTA, sodium tosylchloramide

CE 0123

Application

- 1 % as additive to procedural water
- undiluted for periodical decontamination

Compatibility

Very good compatibility with materials

Clinical data

- Institute Heppeler, phys. + ecolog. compatibility, 1997-10
- Prof. Dr. H.-P. Werner, Schwerin, cytotoxicity test 2012-11

Dosage

Application with internal dosage device:

Fill container with undiluted Alpron and ask your service engineer to adjust the dosage device according to the manufacturer's dilution instruction.

Application with external dosage device:

Mix Alpron with external procedural water (e.g. in BottleCareSystem/Cleanwater-System) at 1 % (Ratio: 10 mL Alpron for 990 mL water) and connect it to the dental unit. In case of chair-side dosage devices ask your service engineer to carry out the adjustment.

Field of application

- Alpron is a concentrate for the continuous chemical conditioning and decontamination of procedural water in DUWL (including instrument tubes and tumbler filler).
- Prevents the regrowth of germs
- Removes and prevents the formation of algae
- Prevents the sedimentation of lime and deposits
- Non-corrosive

Application only by qualified personnel with adequate protective clothing

Shelf life

2 years

Approvals

Approved amongst others by Sirona Dental, Planmeca

Environment

The concentrate is harmful to aquatic life with long lasting effects.

Delivery forms

REF 3184 4 x 1 L bottle in cardboard box (ARTG No. 226413)

MANUFACTURER:

ALPRO MEDICAL GMBH
Mooswiesenstraße 9
78112 St. Georgen/Germany

+49 7725 9392-0
+49 7725 9392-91

Distributor:

**Biodegree Pty Ltd
Sydney, Australia**

info@alpro-medical.de
www.alpro-medical.com