

SURFACE SPRAY DISINFECTANT - HOSPITAL GRADE -
AUST L: 161366

as at: 13 July 2015

'Ready to use' product for quick spray disinfection and cleaning of environmental surfaces and non-medical products in hospitals, dental surgeries, etc. Free of any aldehydes and phenols.

Not to be used on therapeutic devices.

- *Particularly mild*
- *Free of aldehydes and phenols*
- *Pleasant fresh fragrance*

Composition

100 g contain: 45 g ethanol, 15 g isopropanol, 0,1 g chlorhexidine-digluconate, 0,08 g trialkylethoxyammoniumpropionate, 0,05 g alkylamine derivates

Efficacy

bactericidal against Staphylococcus aureus (incl. MRSA), Pseudomonas aeruginosa, Salmonella choleraesuis.

Times of effect (undiluted)

bactericidal in 10 minute

Clinical data

Siliker, Melbourne, AOAC method 961.02 (Germicidal Spray Products as Disinfectant) in accordance with the TGO 54 under 'clean' and 'dirty' conditions, 2008-12

Please feel free to request these clinical data.

Field of application

'Ready to use' product for quick spray disinfection and cleaning of environmental surfaces and non-medical products in hospitals, dental surgeries, etc.

Not to be used on therapeutic devices.

Only for use on alcohol compatible surfaces.

For the cleaning and disinfection of surfaces that are sensitive to alcohol, we recommend the product **PlastiSept**.

Shelf life

3 years



CE 0123

Application

Spray surfaces and objects that are to be disinfected from a distance of approximately 30 cm so that they are completely moistened, and leave the disinfectant to take effect for ten minutes. Afterwards, wipe off with a disposable cloth.

Environment

The active substances in MinutenSpray-classic are biodegradable according to OECD regulations.

Approvals

Approved and recommended amongst others by Sirona Dental

Compatibility

Especially compatible with materials and skin

Delivery forms

REF 3006	500 mL squeeze bottle
REF 4715	1 L spray bottle with spray head
REF 3005	5 L can

Distributor:

**Biodegree Pty Ltd
Sydney, Australia**

Manufacturer:

ALPRO MEDICAL GMBH
Mooswiesenstraße 9
D - 78112 St. Georgen
Germany

+49 7725 9392-0
+49 7725 9392-91

info@alpro-medical.de
www.alpro-medical.com