

INSTRUMENT GRADE DISINFECTANT - LOW LEVEL
READY FOR USE DISINFECTING FOAM
ARTG No: 164389

as at: 5 July 2017 - AU

Alcohol-free solution ready for use for the disinfection of sensitive surfaces of medical devices such as medical and dental units incl. upholstery and suspension tables, X-ray machines, instruments and acrylic glass.



WITHOUT allergenic or sensitizing resp. material damaging active agents as:

- aldehydes
- phenols
- alcohol
- QAC



Composition

Dodecyldipropylenetriamine 0.2 - 1 %
 N-C12-14-Alkyltrimethylenediamine 0.2 – 1 %
 deion. water, fragrances, emulsifying agent

CE 0123

Efficacy/Certification

- bactericidal – e. g. *Staphylococcus aureus* (incl. MRSA), *Pseudomonas aeruginosa*, *Enterococcus hirae*
- yeasticidal – e. g. *Candida albicans*
- virucidal – e. g. *Vaccinia* (representing enveloped viruses acc. to EN standards)
- with remanence effect/preservation (reduction of microbial recontamination)

Clinical data

Prof. Dr. H.-P. Werner, Schwerin:

- VAH*, 2014-05
- EN 13624 + EN 13727, 2014-03
- virucidal (carrier test, acc. to DVV** 2012) against enveloped viruses, 2015-01
- Dr. H. Brill, Hamburg, EN 16615, 2016-01

In special cases clinical data and approvals may be requested from us.

Contact times

yeasticidal	1 min.
bactericidal + MRSA	5 min.
virucidal, Vaccinia	5 min.

Field of application

PlastiSept[®]eco is a wipe disinfectant especially compatible with materials, to use on surfaces of medical devices (such as turbines, contra-angles and hand pieces, unit grips, suspension tables) and to foam susceptible surfaces as surfaces sensitive to alcohol, surfaces of acrylic glass (also for solarium beds) and for users who react sensitively to alcoholic quick disinfectants.

* VAH – Association of Applied Hygiene
 ** DVV – German Association for the Control of Virus Diseases

INSTRUMENT GRADE DISINFECTANT - LOW LEVEL
READY FOR USE DISINFECTING FOAM
ARTG No: 164389

as at: 5 July 2017 - AU

Shelf life

3 years

Application

Moisten surfaces and objects to be disinfected completely and let the disinfectant take effect according to the spectrum of activity/contact times. Afterwards wipe off with a disposable wipe.

Utilization only by qualified personnel wearing protective clothing!

Do not mix with detergents or other chemicals.

Use undiluted.

Environment

The active substances in PlastiSept[®]eco are biodegradable according to OECD regulations. In order to save raw material for the packing and preserve our environment we recommend the use of the 5 litre canister for refilling the foam bottle.

Compatibility

Free of alcohol, aldehydes, QAC and halogen, therefore particularly mild to surfaces

Delivery forms

REF 4645 500 mL foam bottle
REF 4646 5 L canister

Accessories

REF 4010 dispenser tap
REF 4011 dosage pump
REF 4012 can wrench

Distributor:

**Biodegree Pty Ltd
Sydney, Australia**