

Decontamination of procedural water lines in medical and dental units

as at: 6 July 2017 - AU

ARTG No: 226413

Liquid concentrate for the continuous maintenance of the water quality in procedural water lines (including instruments and tumbler filler) of medical and dental units.

- Application of concentrate at 1 %
- Reduces germ load and prevents the regrowth of germs
- Removes and prevents the formation of algae (biofilm)
- Prevents the sedimentation of lime and deposits
- Non-corrosive



CE 0123

Composition

Phenoxyethanol, PHMB (biguanide), EDTA, sodium tosylchloramide

Application

- 1 % as additive to procedural water
- undiluted for periodical decontamination

Clinical data

- Institute Heppeler, phys. + ecolog. compatibility, 1997-10
- Prof. Dr. H.-P. Werner, Schwerin, cytotoxicity test 2012-11

In special cases clinical data and approvals may be requested from us.

Field of application

- Alpron is a concentrate for the continuous chemical conditioning and decontamination of procedural water in DUWL (including instrument tubes and tumbler filler).
- Prevents the regrowth of germs
- Removes and prevents the formation of algae
- Prevents the sedimentation of lime and deposits
- Non-corrosive

Shelf life

2 years

Environment

The ready for use solution is physiologically and ecologically harmless.

MANUFACTURER:
ALPRO MEDICAL GMBH
Mooswiesenstraße 9
78112 St. Georgen/Germany

☎ +49 7725 9392-0
☎ +49 7725 9392-91

Compatibility

Very good compatibility with materials

Dosage

Application with internal dosage device:

Fill container with undiluted Alpron and ask your service engineer to adjust the dosage device according to the manufacturer's dilution instruction.

Application with external dosage device:

Mix Alpron with external procedural water (e.g. in BottleCareSystem/Cleanwater-System) at 1 % (Ratio: 10 mL Alpron for 990 mL water) and connect it to the dental unit. In case of chair-side dosage devices ask your service engineer to carry out the adjustment.

Application only by qualified personnel with adequate protective clothing

Approvals

Approved amongst others by Sirona Dental, Planmeca

Delivery forms

REF 3184 4 x 1 L bottle in cardboard box (ARTG No. 226413)

Distributor:

**Biodegree Pty Ltd
Sydney, Australia**

✉ info@alpro-medical.de
🌐 www.alpro-medical.com