

INSTRUMENT GRADE DISINFECTANT - LOW LEVEL

ARTG no: 164389

as at: 6 July 2015

Alcohol-free disinfectant without any aldehydes 'ready to use'. Especially suitable for surfaces of medical devices which are sensitive to alcohol such as dental units, suspension tables and non-critical instruments.

- Does not cause any change of surface (discolourations and change of structure) as it could happen with liquids containing aldehydes or phenols
- free of aldehydes, phenols and phosphates
- pleasant fragrance
- alcohol-free
- foam spray – no aerosol



Composition

< 5 % cationic tensides, deion. water, fragrance, emulsifying agent. Disinfecting substances: quaternary alkylammonium chlorides

Efficacy

- bactericidal against *Staphylococcus aureus* (incl. MRSA), *Pseudomonas aeruginosa*, *Escherichia coli*, *Proteus vulgaris*, *Salmonella choleraesuis*.
- fungicidal against *Trichophyton mentagrophytes*.
- virucidal against Human Influenza Virus, Herpes simplex virus type 1 (HSV).

Times of effect (undiluted)

bactericidal	in 10 minutes
fungicidal	in 10 minutes
virucidal	in 10 minutes

Clinical data

- Siliker, Melbourne, TGA Test for Hospital grade Disinfectant under 'dirty' conditions (Option B), 2009-02
- Microbiotest, Sterling Virginia, Virucidal Efficacy Test – Human Influenza Virus, 2005-02; Virucidal Efficacy Test Herpes Simplex Virus type 1, 2005-01; AOAC, Germicidal Spray Test using Fungi, 2004-07; AOAC Germicidal Spray Test, 2004-04

Please feel free to request these clinical data.

Shelf life

3 years

Field of application

PlastiSept is a 'ready to use' disinfecting and cleaning agent to be used on surfaces of medical devices such as dental units, suspension tables and non-critical instruments. Especially for particularly sensitive surfaces which are sensitive to alcohol such as seats, head-rests, plastic foil keyboards, chin rests at x-ray system (e. g. OPG), Perspex surfaces (also for solarium). Also for users who are sensitive to alcoholic quick disinfectants.

Application

Cover surfaces and objects to be disinfected completely with foam and leave for ten minutes. Afterwards wipe with a disposable cloth. Do not mix with other detergents or chemicals. Use undiluted.

Environment

The active substances in PlastiSept are biodegradable according to OECD regulations.

Approvals

Approved by Sirona Dental

Compatibility

Free of alcohol and aldehydes – therefore particularly mild to surfaces

Delivery forms

REF 4653	500 mL foam bottle
REF 3063	5 L can

Distributor:

**Biodegree Pty Ltd
Sydney, Australia**