

Preparation of transmission instruments

(turbines, hand pieces and contra-angles)

as at: 20 July 2015

The compressed air gun WL-Blow with integrated air flow limiter is used to blow out, to dry and to support the cleaning of the internal surfaces of hollow part instruments and transmission instruments after the cleaning with WL-clean / WL-cid*. WL-Blow is intended for the use in the processing room and can be connected to the existing oil-free dry air compressor**.

- *drying*
- *supporting the cleaning*

Field of application

Drying and removing of residues of cleaning agents and maintenance products in hollow part instruments and transmission instruments.

Application

Apply for 3 seconds after the use of WL-clean / WL-cid*, afterwards maintain / lubricate according to manufacturer's instructions. If necessary repeat the procedure.


Application only by qualified personnel.

Delivery forms

REF 4160	WL-Blow Set composed of: <ul style="list-style-type: none">1 x compressed air gun (blue)1 x quick connector G ¼"2 x air flow limiter
REF 4162	Air flow limiter
REF 4144	WL-Blow StarterSet ARTG-No: 248472 <ul style="list-style-type: none">2 x WL-clean2 x WL-cid1 x WL-Blow Set1 x Instruction for use

* Please note manufacturer release!

**The connection of a bacterial filter is recommended!

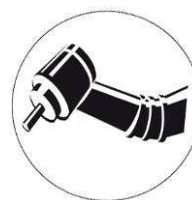

Distributor:

Biodegree Pty Ltd
Sydney, Australia